

## AP-060 SUPERCOR



|                                           |                                                                                                                                                                                                                                                                                                                |
|-------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Product group                             | <b>Abrasives for roughing out</b>                                                                                                                                                                                                                                                                              |
| Product                                   | Corundum in grain                                                                                                                                                                                                                                                                                              |
| Colour                                    | <b>White</b>                                                                                                                                                                                                                                                                                                   |
| Grain size                                | <b>Grain 60</b>                                                                                                                                                                                                                                                                                                |
| Conversion in micron                      | <b>250µ approx.</b>                                                                                                                                                                                                                                                                                            |
| Chemical composition                      | $\text{Al}_2\text{O}_3$ 95,0%<br>$\text{SiO}_2$ 0,4%<br>$\text{FeO}_3$ 0,2%<br>$\text{TiO}_2$ 3,5%<br>$\text{CaO}$ 0,25%<br>$\text{Cr}_2\text{O}_3$ 0,01%                                                                                                                                                      |
| Main features                             | <p>⇒ <b>Hardness, sharp effect, renewing cutting edge and high resistance to wear</b></p> <p>⇒ <b>Most suitable to remove investment and casting oxidations on precious alloys for ceramic or resin</b></p>                                                                                                    |
| Recommended sandblaster                   | <b>Sandblaster for roughing out with abrasive recycle</b>                                                                                                                                                                                                                                                      |
| Aspect of the surface after the treatment | <p>The metal surface after the treatment, if carried out at the correct pressure, is clean, clear and homogeneously rough.</p> <p><b>ATTENTION:</b> the roughness on the surface is originated by the material raised when the abrasive grain hits and penetrates in depth according to the pressure used.</p> |
| Working pressure                          | 3 – 4 BAR (40 - 60 PSI)                                                                                                                                                                                                                                                                                        |
| Recommended nozzle                        | Ø 3,0 mm                                                                                                                                                                                                                                                                                                       |
| Sales package                             | Plastic Can à Kg 5 (code <b>AP-060</b> )<br>Sack à Kg 25 (code <b>AP060S</b> )                                                                                                                                                                                                                                 |